

SALIVA KIT INSTRUCTIONS



Step 1: Contact us

Call **(+44) 020 8068 8176** or send an email to support.uk@igenomix.com to request more information, kits or a collection.



Step 2: Receipt of the Kit

Check that your Saliva Kit includes the following:

- 2 x Saliva tubes (**check expiration date**)
- 2 x White sticky labels
- 'Returning a Sample' instruction sheet
- 2 x Rigid plastic blisters (secondary containers)
- Test Requisition Form (unless one has already been completed electronically)



Step 3: Sample collection

PLEASE CAREFULLY REFER TO THE INSTRUCTIONS INCLUDED WITH THE SALIVA KIT TO PROVIDE YOUR SAMPLE. AS STATED IN THE INSTRUCTIONS:



Do **NOT** eat, drink, smoke, or chew gum for at least **30 minutes** before collecting your sample



Perform the sample collection process **CAREFULLY** and **THOROUGHLY**



REPEAT THE PROCESS WITH BOTH TUBES

INCORRECT SAMPLE COLLECTION MAY LEAD TO SAMPLE FAILURE

Video instructions are available at:

<https://www.dnagenotek.com/ROW/support/collection-instructions/oragene-dna/OG-510andOG-610.html>

Labelling and packaging

- Label each tube with two unique identifiers using the white sticky labels provided:
Patient initials/full name and Date of Birth **OR** Patient initials/full name and Unique Patient ID
- Insert each tube containing the sample into a rigid plastic blister (secondary container)
- Store the samples at room temperature, between 15-25°C; do **NOT** refrigerate or freeze the samples

NEXT: Please refer to the **Returning a Sample** instruction sheet